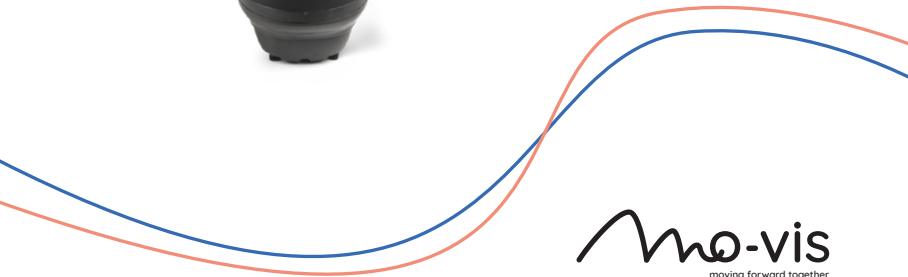
User manual

Edition D-P002-73-70-M9-0800, September 2023

Heavy Duty Joystick

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EN: You can find this manual in English on our website.

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You can find the latest version of all our manuals on the website.

About this manual

User manual

Thank you for choosing a mo-vis product!

This manual contains **useful and important information** about your device.
Please read it carefully before use and store safely for future reference.

Our team (or your authorized dealer) will be happy to answer your questions.

mo-vis bv

Biebuyckstraat 15D . 9850 Deinze .
Belgium

<http://www.mo-vis.com> .
contact@mo-vis.com . +32 9 335 28

60

CH	REP
----	-----

CH-REP: SKS Rehab AG, Im Wyden,
8762 Schwanden GL, Switzerland.

**UK
CA**

UK Responsible Person: QServe
Group UK, Ltd., 49 Greek Street,
London, W1D 4EG, UK.

User manual

Warning labels

Please read this manual, the safety instructions and warning texts carefully, in order to reduce the risks associated to the device. Our products are safe under normal and reasonably foreseeable operating conditions.

NOTE: This symbol indicates general notes and information.

CAUTION: This symbol indicates caution for a hazardous situation that, if not avoided, could result in minor or moderate injury.

WARNING: This symbol indicates a warning for a hazardous situation that, if not avoided, could result in death or serious injury.

Other labels:

Catalogue number: indicates the manufacturer's catalogue number so that the medical device can be

	identified.
	Medical device: indicates that the item is a medical device.
	Date of manufacture: indicates the date when the medical device was manufactured.
	Serial number: indicates the manufacturer's serial number so that a specific medical device can be identified.
	Consult instructions for use or consult electronic instructions for use: indicates the need for the user to consult the instructions for use.
	Keep dry: indicates a medical device that needs to be protected from moisture.
	Do not use if package is damaged and consult instructions for use:

indicates that a medical device should not be used if the package has been damaged or opened and that the user should consult the instructions for use for additional information.

CE label: indicates that the manufacturer or importer affirms the good's conformity with European health, safety, and environmental protection standards.

WEEE: indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.

Manufacturer: indicates the medical device manufacturer.

Support, scrapping & recycling

Technical support

TROUBLE: In case of technical problems:

- Contact your dealer
- If your dealer is not available or unknown, please contact mo-vis: contact@mo-vis.com or +32 9 335 28 60.

Always state the product code and device serial number when contacting us. This ensures you are provided with the correct information.

Spare parts and accessories

Contact mo-vis or your dealer for more information about spare parts and accessories.

Scrapping & recycling

The expected lifetime of this device is 5 years.

CAUTION: For scrapping, adhere to your local waste legislation. Dispose of obsolete electronic parts responsibly in accordance with local recycling regulations.

Limited liability

*mo-vis accepts **no liability** for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings and instructions in this manual.*

CAUTION: This product should only be installed by a qualified service engineer.

NOTE: All mo-vis user manuals can be found at <http://www.mo-vis.com> where they can be consulted in PDF format.

NOTE: In case any serious incident occurs in relation to this device, this should be reported immediately to mo-vis and the competent authority of the Member State in which the user is established.

Intended Use

The Heavy Duty Joystick:

- is a joystick module which can be connected directly to the wheelchair electronics to control the wheelchair functions

- is a large format, extremely durable, proportional joystick for all-round heavy duty use
- can be used as a hand or foot operated joystick
- has been developed for users with excessive force (spasticity, dystonia...)

Features

The Heavy Duty Joystick is:

- Fully proportional with increased throw (40 mm - 1.57 in) and force (ca. 650 gr - 22.93 oz)
- built in a compact housing, with two 3.5 mm jacks for switch input: (1) red = on/off power wheelchair - (2) yellow = mode of the power wheelchair.
- is fully adjustable to individual possibilities and needs with well-thought-out electronics (USB-C connection).
- is compatible with multiple types of wheelchair electronics.
- LED light in the housing which will start to flash in case of error

Operation

The movements of the control are translated into according movements of the wheelchair, e.g. driving or menu navigation.

Common practice to navigate the wheelchair with the joystick is as follows:

- **Direction:** point the control into the direction you want the wheelchair to move. The wheelchair then moves in that direction.
- **Speed:** the further you move the control from the default (center) position, the faster the wheelchair moves.
- **Stop:** whenever you release the control, the control moves back to the default (center) position and the wheelchair stops.

CAUTION: Avoid hitting obstacles during driving.

WARNING: When the LED light flashes and/or after every incident with the wheelchair or the mo-vis device, contact your dealer immediately to perform a functional test.

LED status

The illumination of the LED on the joystick interface indicates the operational status of the joystick.

JOYSTICK STATUS	COLOUR	LED TIMING	COUNT	STATE DESCRIPTION
Configuring	Orange	Fast	Not applicable	Wheelchair is configuring
Powercycling	Orange	Heartbeat		Wheelchair waits for

JOYSTICK STATUS	COLOUR	LED TIMING	COUNT	STATE DESCRIPTION
cle				power cycle
Focus	Green	Always on		System is in focus
Out of focus	Green	Heartbeat		System is out of focus
Out of neutral	Green	Medium		Joystick is out of neutral
Error	Red	Errors	See	Something went wrong

Parts and accessories

Package

The Heavy Duty Joystick package consists of:

EXAMPLE	P002-73 HEAVY DUTY JOYSTICK OMNI	P002-77 HEAVY DUTY JOYSTICK R-NET
	Heavy Duty Joystick set with integrated SUB D9 cable and 2 M6*12	Heavy Duty Joystick set with integrated R-net cable and 2 M6*12

EXAMPLE	P002-73 HEAVY DUTY JOYSTICK OMNI	P002-77 HEAVY DUTY JOYSTICK R-NET
	bolts for mounting	bolts for mounting

D-P002-73-70-MX Heavy Duty Joystick User Manual

D-P002-73-70-M1 Heavy Duty Joystick Installation Manual

Accessories

For mounting and further personalization, the following accessories are available:

EXAMPLE	PRODUCT DESCRIPTION
	M002-28 Heavy Duty Joystick

EXAMPLE

PRODUCT DESCRIPTION

mounting ring assembly

Unit detail

- The connections of the unit have protective covers to avoid the intrusion of dust or moist during usage.
- When in use, a green LED on the joystick indicates the proper functioning of the unit. In case of an error, the LED light will flash. See the *Installation Manual* for more information.

Spare parts

Spare parts and accessories must be ordered by the dealer at mo-vis bv.
Always mention the product code of the desired item.

Qualifications

CAUTION: Only a qualified service engineer may install the device.

CAUTION: An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

First time use

CAUTION: A qualified service engineer should assist during first time use of the device and make sure that all functions are clear to the user, including what to do in case of problems or doubt.

Cleaning

WARNING: As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

- Gently remove dust and dirt with a damp cloth.
- Use only non-aggressive disinfectant cleaning agents.

CAUTION: Do not immerse in water or use excessive amounts of liquid.

CAUTION: Do not apply additional lubricants to the moving parts.

The handles of the joystick can be removed to clean them.

- You may shortly immerse the handle in warm water with a gentle cleaning agent. Let the handle dry before putting it back on.
- If the handle is damaged, worn out or remains dirty, please contact your authorized dealer or mo-vis to replace it.

Warnings and limitations

NOTE: This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

WARNING: Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

CAUTION: If an output inhibit is programmed, then it should be checked monthly if this is still operational. Please see the *Installation Manual* for more information.

Connection options

You can connect 1 or 2 switches to the unit:

- **Red** = power on/off
- **Yellow** = mode

WARNING: The on/off switch must be available to the user at all times. This allows to instantly stop the wheelchair in case of problems or an emergency.

WARNING: Always power off the device when the user is no longer in the wheelchair to avoid unauthorized access or accidental movements.

Combinations

There are different versions of this device available for combination with different wheelchair electronics. Please refer to the *Installation Manual* of this device for more information.

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