User manual

Edition D-P034-71-70-M9-0100, October 2025

R-net Switch Control

P034-71 R-net Switch Control

Contents

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EN: You can find this manual in English on our website.

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You can find the latest version of all our manuals on the website.

About this manual

User manual

Thank you for choosing a mo-vis product!

This manual contains **useful and important information** about your device. **Please read it carefully before use and store safely for future reference.**

Our team (or your authorized dealer) will be happy to answer your questions.

mo-vis bv

Biebuyckstraat 15D . 9850 Deinze . Belgium
<http://www.mo-vis.com>
contact@mo-vis.com
+32 9 335 28 60

CH-REP: SKS Rehab AG, Im Wyden, 8762
Schwanden GL, Switzerland.

UK Responsible Person: QServe Group UK,
Ltd., 49 Greek Street, London, W1D 4EG,
UK.

User manual

Warning labels

Please read this manual, the safety instructions and warning texts carefully, in order to reduce the risks associated to the device. Our products are safe under normal and reasonably foreseeable operating conditions.

NOTE: This symbol indicates general notes and information.

CAUTION: This symbol indicates caution for a hazardous situation that, if not avoided, could result in minor or moderate injury.

WARNING: This symbol indicates a warning for a hazardous situation that, if not avoided, could result in death or serious injury.

Other labels:

	Catalogue number: indicates the manufacturer's catalogue number so that the medical device can be identified.
	Batch code: indicates the manufacturer's batch code so that the batch or lot can be identified.
	Medical device: indicates that the item is a medical device.
	Date of manufacture: indicates the date when the medical device was manufactured.
	Serial number: indicates the manufacturer's serial number so that a specific medical device can be identified.
	Consult instructions for use or consult electronic instructions for use: indicates the need for the user to consult the instructions for use.

	Keep dry: indicates a medical device that needs to be protected from moisture.
	Do not use if package is damaged and consult instructions for use: indicates that a medical device should not be used if the package has been damaged or opened and that the user should consult the instructions for use for additional information.
	CE label: indicates that the manufacturer or importer affirms the good's conformity with European health, safety, and environmental protection standards.
	WEEE: indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
	Manufacturer: indicates the medical device manufacturer.

Support, scrapping & recycling

Technical support

TROUBLE: In case of technical problems:

- Contact your dealer
- If your dealer is not available or unknown, please contact mo-vis: contact@mo-vis.com or +32 9 335 28 60.

Always state the product code and device serial number when contacting us. This ensures you are provided with the correct information.

Spare parts and accessories

Contact mo-vis or your dealer for more information about spare parts and accessories.

Scrapping & recycling

The expected lifetime of this device is 5 years.

CAUTION: For scrapping, adhere to your local waste legislation. Dispose of obsolete electronic parts responsibly in accordance with local recycling regulations.

Limited liability

mo-vis accepts **no liability** for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings and instructions in this manual.

CAUTION: This product should only be installed by a qualified service engineer.

NOTE: All mo-vis user manuals can be found at <http://www.mo-vis.com> where they can be consulted in PDF format.

NOTE: In case any serious incident occurs in relation to this device, this should be reported immediately to mo-vis and the competent authority of the Member State in which the user is established.

Intended Use

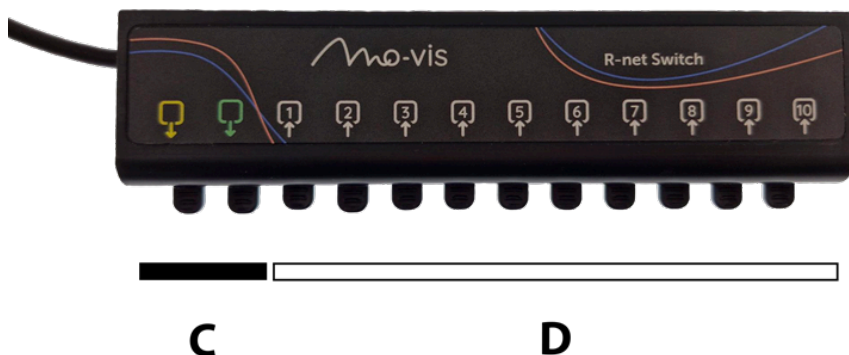
The R-net Switch Control is designed to enhance the accessibility and control for power wheelchair users who find standard keypads or multitude of buttons challenging.

By integrating a **custom-made switch array**, users can efficiently manage both **driving and auxiliary wheelchair functions**, such as direction, lighting, and other essential controls.

It is particularly beneficial for individuals who wish to activate these functions seamlessly **while driving**, thereby promoting independence and safety.

Features

The R-net Switch Control (picture above) consists of an P002-86 Joystick Interface R-net Switch (A) and a P034-50 10 Switch Box (B) which contains 10 mono jack connections for inputs (D) and 2 mono jack connections for outputs (C) (yellow and green).

The jacks have default settings that can be checked and adjusted by the dealer in the mo-vis Configurator Software. For more information see the *Installation Manual*.

- To the jacks for input (D) you can connect any button with 3.5 mm jack connection. Triple action programmability is possible for each button: direct action OR short/medium/long action (with auditive feedback). This minimizes the number of buttons needed.
- Auxiliary functions **while driving**: mode selection, profile changes, lighting, horn and mouse control

NOTE: These auxiliary functions can be operated independently of the device in focus and modeless. The only exception here is mouse control - for that you need to be in Bluetooth mode.

- Driving and actuator functions

WARNING: If the function of a certain button is driving or actuator changes, you should use our Twister Pro (or another button with resistor values). These can be configured as safety switches using an integrated dip switch. For more information please refer to the *User Manual* and/or *Installation Manual* of the Twister Pro. If you use a regular 'normal open' button, such as our Twisters, then you should only execute auxiliary wheelchair functions (e.g. lighting, horn...).

- To the jacks for output (C) you can connect any output that you could otherwise also operate with a button. This way, you can control devices in your environment.

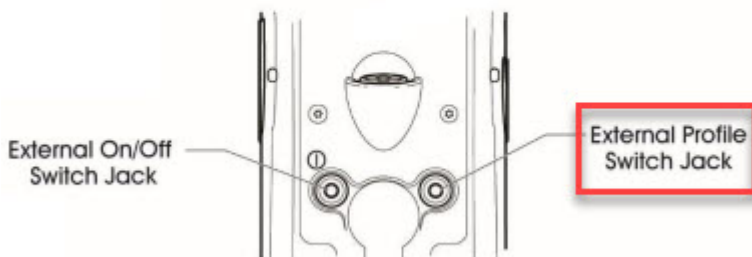
 **NOTE:** The outputs are isolated from the wheelchair electronics. Each output is able to handle a maximum continuous current of 75 mA and a peak current of 350 mA.

 **CAUTION:** The R-net Switch Control does not contain a double check against component failure. So the outputs cannot be used for safety-critical purposes.

Some interesting features of the R-net Switch Control:

- Multiple buttons can be activated at the same time.
- Actuators can be controlled in toggle mode.
- Combi seat functions can be controlled by a button.
- Smooth driving with switches, thanks to the Shaping parameter. See *Installation Manual*.
- You don't need a Curtiss-Wright Omni(2) to drive with switches.
- Joystick drivers who have difficulty driving into or out of an elevator/car can use the buttons of the R-net Switch Control to only drive straight forward/backward.
- •

EXTERNAL JACK SOCKETS

With the R-net Switch Control you can have multiple functions. A few examples:

- Example 1: Connect 1 button to the **External Profile Switch Jack** of the JSM for the function **Profile/mode select** + Connect 1 button to the first input jack of the R-

net Switch Control for the function **Bluetooth external switch**.

Figure 1: Example 1

- Example 2: Connect 1 button to the first input jack and 1 button to the second input jack of the R-net Switch Control and assign the first the function **External Profile Switch Jack** and the second **Bluetooth external switch**.
-

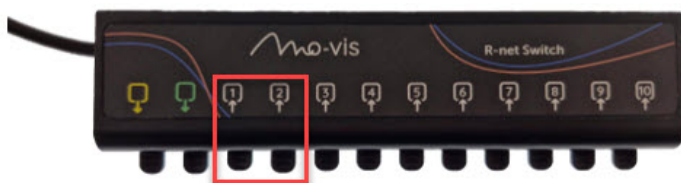
Figure 2: Example 2

- Example 3: Connect 1 button to the first input jack of the R-net Switch Control and assign the function **External Profile Switch Jack** to the short press and **Bluetooth**

external switch to the medium press of that button.

-

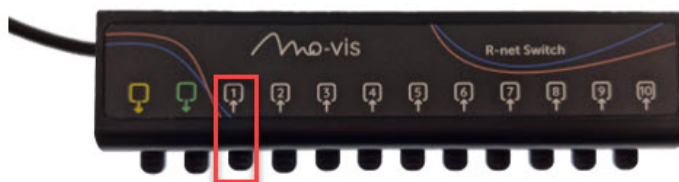
Figure 3: Example 3

- Without a R-net Switch Control you can use your joystick to cycle through the different Bluetooth devices connected to the wheelchair. However, if you then want to perform a click, you have to do so with a nudge. For some users, this is difficult. With the R-net Switch Control, you can perform that click as well with a button.

Operation

The activation of certain switches connected to the control is translated into certain actions, e.g. driving or auxiliary functions.

Common practice to navigate the wheelchair with the P034-71 R-net Switch Control is as follows:

- **Direction:** a button can be allocated to a driving direction. Pressing that button, will lead to a movement of the wheelchair in that direction as long as you hold the button.
- **Speed:** there is only 1 speed (fastest speed of the profile).
- **Stop:** whenever you release the button, the wheelchair stops driving.

 **CAUTION:** Avoid hitting obstacles during driving.

 **WARNING:** Contact your dealer immediately to perform a functional test in the following situations:

- When the interface indicates an error code
- When you hear an error/warning beep
- After every incident with the wheelchair

CAUTION: Before inserting a connector, remove the protective cover. If the connections are not used, always put or keep the protective covers in.

LED status

The illumination of the LED on the P002-86 Joystick Interface R-net Switch indicates the operational status of the device.

JOYSTICK STATUS	COLOUR	LED TIMING	COUNT	STATE DESCRIPTION
Configuring	Orange	Fast	Not applicable	Wheelchair is configuring
Powercycle	Orange	Heartbeat		Wheelchair waits for power cycle
Focus	Green	Always on		System is in focus
Out of focus	Green	Heartbeat		System is out of focus
Out of neutral	Green	Medium		Device is out of neutral
Error	Red	Errors	See <i>Installation Manual</i>	Something went wrong
Warning	Orange	Warning	See <i>Installation Manual</i>	Something went wrong

Auditive feedback

There is also auditive feedback available, that indicates a state change. Each state has its own sequence.

Parts and accessories

The R-net Switch Control package (P034-71) consists of the following elements:

1	P034-50 10 Switch Box
2	P002-86 Joystick Interface R-net Switch
3	<i>D-P034-71-70-MX R-net Switch Control User manual</i>
4	<i>D-P034-71-70-M9 R-net Switch Control Installation manual</i>

NOTE: There are no mounting screws included in the package as we don't know where it will be mounted. We recommend using d:3mm plastic screws. Depending on where you mount the P034-50, it is recommended to adjust the screw length. Please refer to [Dimensions on page 17](#) for more information on the position and dimensions of the mounting holes.

Optional

We recommend using a Twister Pro for driving and actuator functions. For more information about the Twister Pro and the mounting options, please refer to the *User Manual* and *Installation Manual* of that device.

We also have a variety of 'normal open' Twisters available, that can be used for the auxiliary wheelchair functions. For more information on these devices, please refer to the *User Manual* of the Twister.

Qualifications

CAUTION: Only a qualified service engineer may install the device.

CAUTION: An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

First time use

CAUTION: A qualified service engineer should assist during first time use of the device and make sure that all functions are clear to the user, including what to do in case of problems or doubt.

Cleaning

Regular maintenance of the device is required. Before you do this, always disconnect the device.

- Gently remove dust and dirt with a damp cloth.
- Use only non-aggressive disinfectant cleaning agents.

CAUTION: Do not immerse in water or use excessive amounts of liquid.

CAUTION: Make sure the device and its buttons stay clean, so activation remains possible at all times.

CAUTION: If the device or a connected button is damaged, it must be replaced immediately to ensure correct functionality.

R-net Switch Control Connection

The R-net Switch Control includes a Joystick Interface R-net Switch.

That interface has an R-net bus cable.

You can connect 1 or 2 switches to the unit:

- **Red** = power on/off
- **Yellow** = profile/mode

WARNING: The on/off switch must be available to the user at all times. This allows to instantly stop the wheelchair in case of problems or an emergency.

WARNING: Always power off the device when the user is no longer in the wheelchair to avoid unauthorized access or accidental movements.

For more information on the connection and installation, please refer to the *Installation Manual*.

Warnings and limitations

NOTE: This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

WARNING: Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

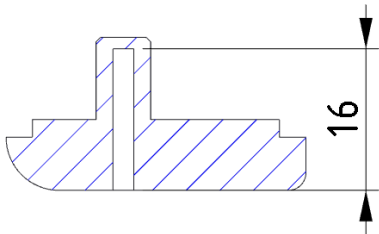
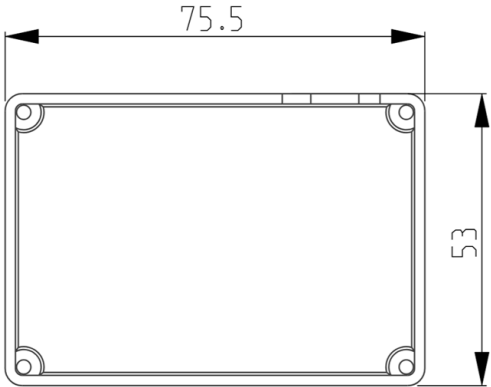
CAUTION: If an output inhibit is programmed, then it should be checked monthly if this is still operational. Please see the *Installation Manual* for more information.

Dimensions

NOTE: All dimensions below are in mm.

DESCRIPTION	DRAWING
Dimensions 10 Switch Box - front view	
Dimensions 10 Switch box - side view	
Placement of the mounting holes on the 10 Switch Box	
Dimensions of the mounting holes on the 10 Switch Box - use d:3mm plastic screws	

DESCRIPTION	DRAWING
Depth of the mounting holes on the 10 Switch Box - use d:3mm plastic screws	 A cross-sectional drawing of a mounting bracket. The bracket has a base with a semi-circular end on the left and a vertical post on the right. A dimension line with arrows at both ends indicates a height of 16 from the base to the top of the vertical post. The bracket is filled with blue diagonal hatching.
Dimensions of the Joystick Interface R-net Switch	 A top-down view drawing of a rectangular joystick interface. The rectangle has rounded corners and a double-line border. Four small circles, representing screws, are located at each corner. Dimension lines with arrows indicate a width of 75.5 and a height of 53.
Cable length between P034-50 and P002-86 is 1500 mm	

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UPOZORNĚNÍ: Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

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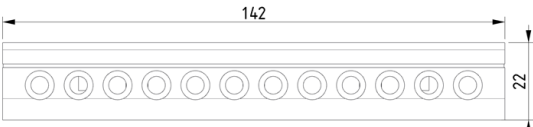
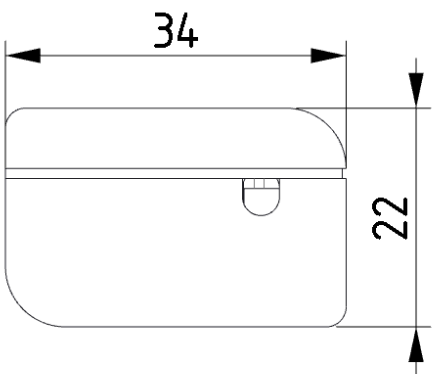
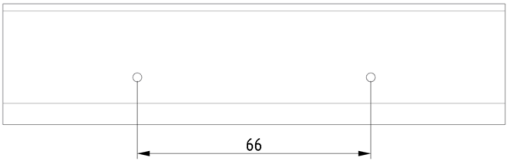
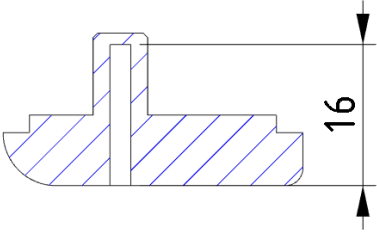
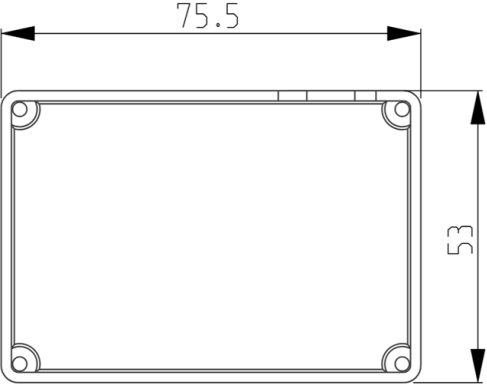
POPIS	VÝKRES
Rozměry rozhraní Joystick Interface R-net Switch	 The drawing shows a rectangular frame with rounded corners. The horizontal dimension is labeled 75.5 and the vertical dimension is labeled 53. The frame has a double-line border. At each of the four corners, there is a small circular feature, possibly a screw or a mounting point. The top edge of the frame has a small rectangular protrusion near the right corner.
Délka kabelu mezi jednotkami P034-50 a P002-86 je 1 500 mm	

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WARNUNG: Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

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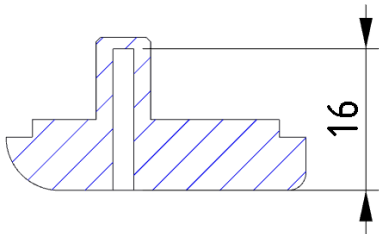
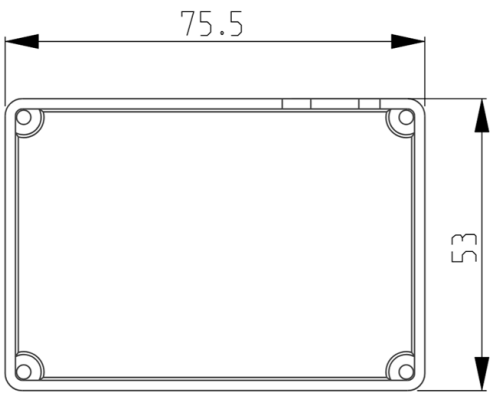
BESCHREIBUNG	ZEICHNUNG
Tiefe der Montagebohrungen an der 10 Switch Box - 3-mm-Plastikschrauben verwenden	 A technical cross-section drawing of a component, likely a switch box. It shows a base with a central vertical slot and side flanges. A dimension line on the right indicates a depth of 16 mm from the base to the top of the central slot.
Abmessungen des Joystick R-net Interface Switch	 A technical top-down drawing of a rectangular component. It features a double-line border and rounded corners. Dimension lines indicate a width of 75.5 mm and a height of 53 mm.
Die Kabellänge zwischen P034-50 und P002-86 beträgt 1.500 mm	

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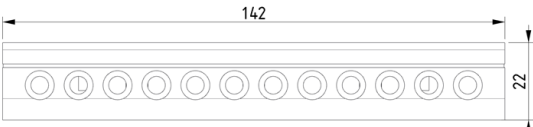
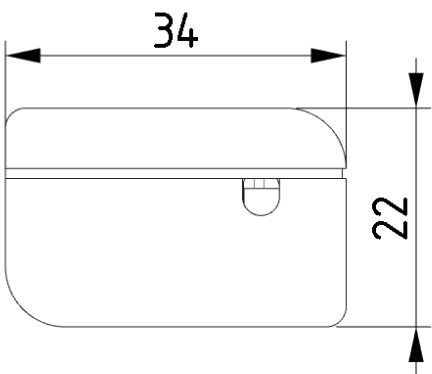
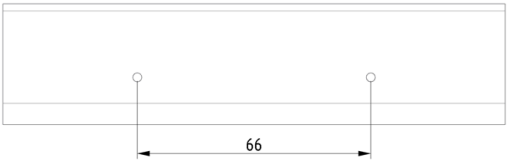
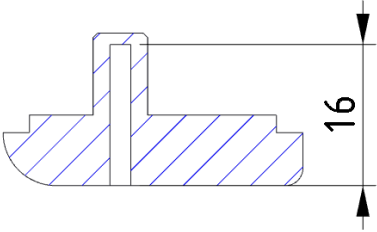
 **AVISO:** Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

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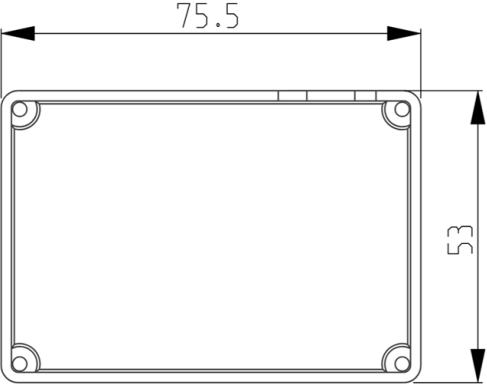
DESCRIPCIÓN	ESQUEMA
Dimensiones del interruptor Joystick R-net Interface	 The diagram shows a rectangular component with rounded corners. A horizontal dimension line at the top indicates a width of 75.5 mm. A vertical dimension line on the right indicates a height of 53 mm. The component has four circular features at the corners, likely for mounting or connection. There are also small rectangular features along the top edge.
La longitud del cable entre P034-50 y P002-86 es de 1.500 mm	

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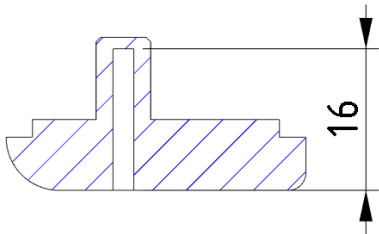
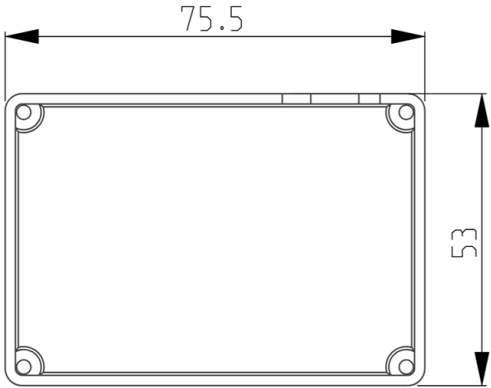
 **ATTENTIE:** Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

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BESCHRIJVING	TEKENING
Diepte van de bevestigingsgaten in het 10 schakelkastje - gebruik kunststof schroeven, d=3 mm	 A technical cross-section drawing of a switch housing. It shows a base with a central vertical slot and a raised section on the right. A dimension line with arrows at both ends indicates a depth of 16 mm from the base to the top of the raised section.
Afmetingen van de joystick R-net interfaceschakelaar	 A technical top-down drawing of a rectangular joystick interface switch. The drawing shows a rectangular frame with rounded corners and four circular mounting points at the corners. Dimension lines indicate a width of 75.5 mm and a height of 53 mm.
De kabellengte tussen P034-50 en P002-86 is 1.500 mm	

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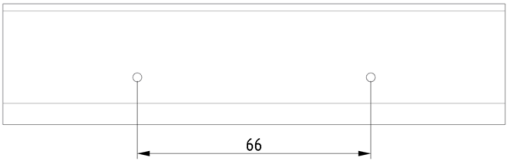
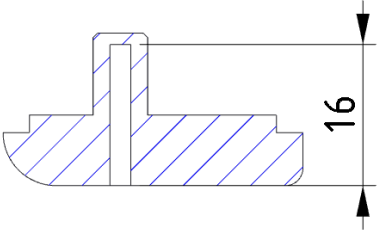
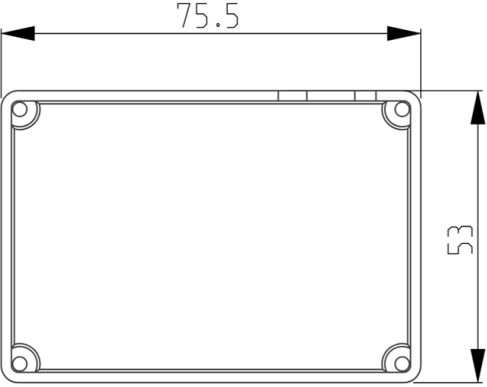
DESCRIÇÃO	DESENHO
Posicionamento dos orifícios de montagem na 10 Switch Box	 A top view diagram of a rectangular component. Two small circles representing mounting holes are positioned horizontally. A dimension line with arrows at both ends indicates the distance between the centers of these two holes, which is labeled as 66.
Dimensões dos orifícios de montagem na 10 Switch Box - utilize parafusos de plástico d:3mm	 A top view diagram of the same rectangular component. Two small circles representing mounting holes are shown. A dimension line with an arrow points from the center of the left hole to a label 'R1,25', indicating the radius of the hole.
Profundidade dos orifícios de montagem na 10 Switch Box - utilize parafusos de plástico d:3mm	 A cross-sectional diagram of the component. It shows a stepped profile with a central vertical slot. The depth of the holes is indicated by a vertical dimension line on the right side, with arrows at the top and bottom, labeled with the number 16.

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