

# User manual

Edition D-P002-71-70-M0-0902, October 2024

## All-round (Light) Joystick

All-round Joystick Omni (P002-71) - All-round Joystick R-net (P002-75) -  
All-round Joystick Omni Light (P002-72) - All-round Joystick R-net Light  
(P002-76)



# Contents

<b>Other languages .....</b>	<b>2</b>
<b>About this manual .....</b>	<b>3</b>
User manual .....	3
mo-vis bv .....	3
<b>User manual .....</b>	<b>4</b>
Warning labels .....	4
Support, scrapping & recycling .....	5
Limited liability .....	6
Intended use .....	6
Features .....	6
Operation .....	7
Parts and accessories .....	8
Qualifications .....	10
First time use .....	11
Cleaning .....	11
Warnings and limitations .....	11
Connection options .....	12
Combinations .....	12
Dimensions .....	12

## Other languages

CS: Tito příručku naleznete v češtině na našich webových stránkách.

DA: Du kan finde denne manual på dansk på vores websted.

DE: Auf unserer Website finden Sie diese Anleitung in Deutsch.

EN: You can find this manual in English on our website.

ES: Este manual está disponible en español en nuestro sitio web.

FR: Vous pouvez trouver ce manuel en français sur notre site Web.

IT: nostro sito web è disponibile il presente manuale in italiano.

NO: Du finner denne håndboken på norsk på nettstedet vårt.

NL: De Nederlandse handleiding kunt u op onze website vinden.

PT: Pode encontrar este manual em português na nossa página web.

SV: Du hittar den här handboken på svenska på vår webbplats.

**You can find the latest version of all our manuals on the website.**

# About this manual

## User manual

Thank you for choosing a mo-vis product!



This manual contains **useful and important information** about your device. **Please read it carefully before use and store safely for future reference.**

Our team (or your authorized dealer) will be happy to answer your questions.

## mo-vis bv

	Biebuyckstraat 15D . 9850 Deinze . Belgium <a href="http://www.mo-vis.com">http://www.mo-vis.com</a> <a href="mailto:contact@mo-vis.com">contact@mo-vis.com</a> +32 9 335 28 60		
<table border="1"><tr><td data-bbox="126 730 199 778">CH</td><td data-bbox="199 730 272 778">REP</td></tr></table>	CH	REP	CH-REP: SKS Rehab AG, Im Wyden, 8762 Schwanden GL, Switzerland.
CH	REP		
	UK Responsible Person: QServe Group UK, Ltd., 49 Greek Street, London, W1D 4EG, UK.		

# User manual

## Warning labels

*Please read this manual, the safety instructions and warning texts carefully, in order to reduce the risks associated to the device. Our products are safe under normal and reasonably foreseeable operating conditions.*



**NOTE:** This symbol indicates general notes and information.







**CAUTION:** This symbol indicates caution for a hazardous situation that, if not avoided, could result in minor or moderate injury.



**WARNING:** This symbol indicates a warning for a hazardous situation that, if not avoided, could result in death or serious injury.

Other labels:

	Catalogue number: indicates the manufacturer's catalogue number so that the medical device can be identified.
	Batch code: indicates the manufacturer's batch code so that the batch or lot can be identified.
	Medical device: indicates that the item is a medical device.
	Date of manufacture: indicates the date when the medical device was manufactured.
	Serial number: indicates the manufacturer's serial number so that a specific medical device can be identified.
	Consult instructions for use or consult electronic instructions for use: indicates the need for the user to consult the instructions for use.

	Keep dry: indicates a medical device that needs to be protected from moisture.
	Do not use if package is damaged and consult instructions for use: indicates that a medical device should not be used if the package has been damaged or opened and that the user should consult the instructions for use for additional information.
CE	CE label: indicates that the manufacturer or importer affirms the good's conformity with European health, safety, and environmental protection standards.
	WEEE: indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
	Manufacturer: indicates the medical device manufacturer.

## Support, scrapping & recycling

### Technical support



**TROUBLE:** In case of technical problems:

- Contact your dealer
- If your dealer is not available or unknown, please contact mo-vis: [contact@mo-vis.com](mailto:contact@mo-vis.com) or +32 9 335 28 60.

Always state the product code and device serial number when contacting us. This ensures you are provided with the correct information.

### Spare parts and accessories

Contact mo-vis or your dealer for more information about spare parts and accessories.

### Scrapping & recycling

The expected lifetime of this device is 5 years.



**CAUTION:** For scrapping, adhere to your local waste legislation. Dispose of obsolete electronic parts responsibly in accordance with local recycling regulations.

## Limited liability

*mo-vis* accepts **no liability** for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings and instructions in this manual.



**CAUTION:** This product should only be installed by a qualified service engineer.



**NOTE:** All *mo-vis* user manuals can be found at <http://www.mo-vis.com> where they can be consulted in PDF format.



**NOTE:** In case any serious incident occurs in relation to this device, this should be reported immediately to *mo-vis* and the competent authority of the Member State in which the user is established.

## Intended use

The All-round Joystick:

- is a joystick module which can be connected directly to the wheelchair electronics to control the wheelchair functions.
- is the compact version of a standard electronic wheelchair joystick module.
- is developed for all-round use.
- is suitable for the major part of wheelchair users.
- can be used as a standard joystick, chin joystick or as an attendant joystick.

The All-round Joystick Light has the same specifications, but with medium force (see ).

## Features

The All-round Joystick and All-round Joystick Light:

- is fully proportional.

	THROW	FORCE
All-round Joystick	20 mm - 0.8 in	250 gr - 8.82 oz
All-round Joystick Light	7 mm - 0.3 in	120 gr - 4.23 oz

- built in a compact housing, with two 3.5 mm jacks for switch input: (1) red = on/off power wheelchair - (2) yellow = mode of the power wheelchair.




- is fully adjustable to individual possibilities and needs with well-thought-out electronics (USB-C connection).
- is compatible with multiple types of wheelchair electronics.
- equipped with a LED light in the housing which will start to flash in case of error.
- has the possibility to insert 1 or 2 Satellite Twisters via integrated insert parts.


## Operation

The movements of the control are translated into according movements of the wheelchair, e.g. driving or menu navigation.

Common practice to navigate the wheelchair with the joystick is as follows:

- **Direction:** point the control into the direction you want the wheelchair to move. The wheelchair then moves in that direction.
- **Speed:** the further you move the control from the default (center) position, the faster the wheelchair moves.
- **Stop:** whenever you release the control, the control moves back to the default (center) position and the wheelchair stops.

 **CAUTION:** Avoid hitting obstacles during driving.

 **WARNING:** When the LED light flashes and/or after every incident with the wheelchair or the mo-vis device, contact your dealer immediately to perform a functional test.

### LED status


The illumination of the LED on the joystick interface indicates the operational status of the joystick.

JOYSTICK STATUS	COLOUR	LED TIMING	COUNT	STATE DESCRIPTION
Configuring	Orange	Fast	Not applicable	Wheelchair is configuring
Powercycle	Orange	Heartbeat		Wheelchair waits for power cycle
Focus	Green	Always on		System is in focus
Out of focus	Green	Heartbeat		System is out of focus
Out of neutral	Green	Medium		Joystick is out of neutral
Error	Red	Errors	See	Something went wrong

## Parts and accessories

### Package

The All-round Joystick (Light) package consists of:




EXAMPLE	P002-71 ALL-ROUND JOYSTICK OMNI AND P002-72 ALL-ROUND JOYSTICK OMNI LIGHT	P002-75 ALL-ROUND JOYSTICK R-NET AND P002-76 ALL-ROUND JOYSTICK R-NET LIGHT
	All-round Joystick (Light) set, with integrated SUB D9 cable	All-round Joystick (Light) set, with integrated R-net cable

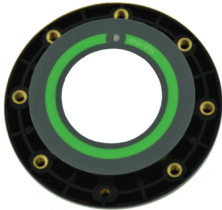

*D-P002-71-70-MX All-round Joystick User manual*

*D-P002-71-70-M1 All-round Joystick Installation manual*

### Accessories

For mounting and further personalization, the following accessories are available:

EXAMPLE	PRODUCT DESCRIPTION
 <p>The image shows the components of the M002-60 All-round Joystick mounting set 8 mm. It includes two black circular mounting plates, each with four mounting holes. There are four silver-colored screws and four silver-colored nuts. A black cylindrical sleeve is also included.</p>	<p>M002-60 All-round Joystick mounting set 8 mm</p>
 <p>The image shows the components of the M002-61 All-round Joystick mounting set 6 mm. It includes two black circular mounting plates, each with four mounting holes. There are four silver-colored screws and four silver-colored nuts. A black cylindrical sleeve is also included.</p>	<p>M002-61 All-round Joystick mounting set 6 mm</p>
 <p>The image shows a single black circular joystick ball, which is the M002-76 All-round Joystick ball (6.35 mm).</p>	<p>M002-76 All-round Joystick ball (6.35 mm)</p>

EXAMPLE	PRODUCT DESCRIPTION
	<p>M002-25 All-round Joystick table mounting set</p>
	<p>P005-40 Satellite Twister (incl. All-round Joystick housing satellite insert), short cable (300 mm), black</p> <p>Available in other colors (40 = black, 41 = red, 42 = yellow, 43 = blue, 44 = green)</p>

### Unit detail

- The connections of the unit have protective covers to avoid the intrusion of dust or moist during usage.
- When in use, a green LED on the joystick indicates the proper functioning of the unit. In case of an error, the LED light will flash. See the *Installation Manual* for more information.

### Spare parts

Spare parts and accessories must be ordered by the dealer at mo-vis bv.


Always mention the product code of the desired item.

### Qualifications


**⚠ CAUTION:** Only a qualified service engineer may install the device.

**⚠ CAUTION:** An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.


## First time use


 **CAUTION:** A qualified service engineer should assist during first time use of the device and make sure that all functions are clear to the user, including what to do in case of problems or doubt.

## Cleaning

 **WARNING:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

- Gently remove dust and dirt with a damp cloth.
- Use only non-aggressive disinfectant cleaning agents.


 **CAUTION:** Do not immerse in water or use excessive amounts of liquid.


 **CAUTION:** Do not apply additional lubricants to the moving parts.


The handles of the joystick can be removed to clean them.

- You may shortly immerse the handle in warm water with a gentle cleaning agent. Let the handle dry before putting it back on.
- If the handle is damaged, worn out or remains dirty, please contact your authorized dealer or mo-vis to replace it.

## Warnings and limitations

 **NOTE:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

 **WARNING:** Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

 **CAUTION:** If an output inhibit is programmed, then it should be checked monthly if this is still operational. Please see the *Installation Manual* for more information.

## Connection options

You can connect 1 or 2 switches to the unit:



- **Red** = power on/off
- **Yellow** = profile/mode

**⚠ WARNING:** The on/off switch must be available to the user at all times. This allows to instantly stop the wheelchair in case of problems or an emergency.

**⚠ WARNING:** Always power off the device when the user is no longer in the wheelchair to avoid unauthorized access or accidental movements.

## Combinations

There are different versions of this device available for combination with different wheelchair electronics. Please refer to the *Installation Manual* of this device for more information.

## Dimensions

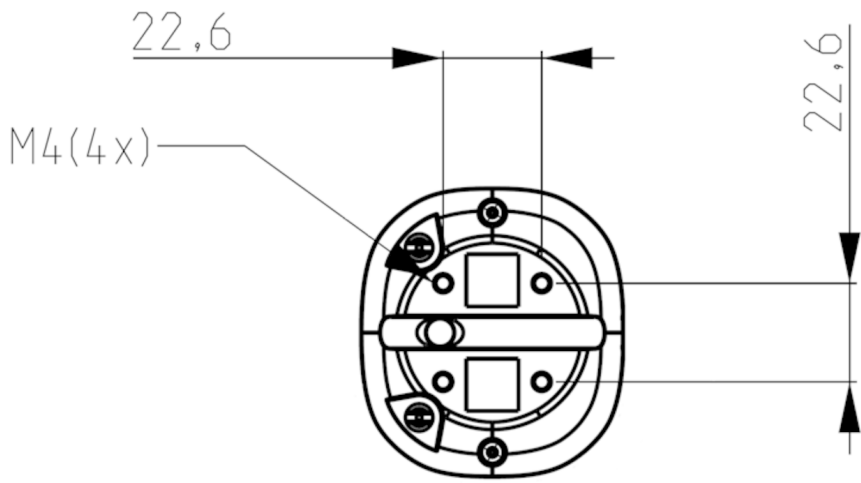
### *All-round Joystick Omni/R-net*

110 x 66 x 60 mm / 4.33 x 2.60 x 2.36 in (HxWxD)

### *All-round Joystick Omni/R-net Light*

96.5 x 66 x 60 mm / 4.33 x 2.60 x 2.36 in (HxWxD)

### *Mounting holes*





mo-vis bv . Biebuyckstraat 15D . 9850 Deinze - Belgium  
www.mo-vis.com . contact@mo-vis.com . +32 9 335 28 60

**Go to our website for more information on our products or share  
your experience with us via email.**

